
Contour[®] plus ONE[»]

Blood Glucose
Monitoring System

Using the
smartLIGHT™ feature on the
CONTOUR®PLUS ONE
smart meter

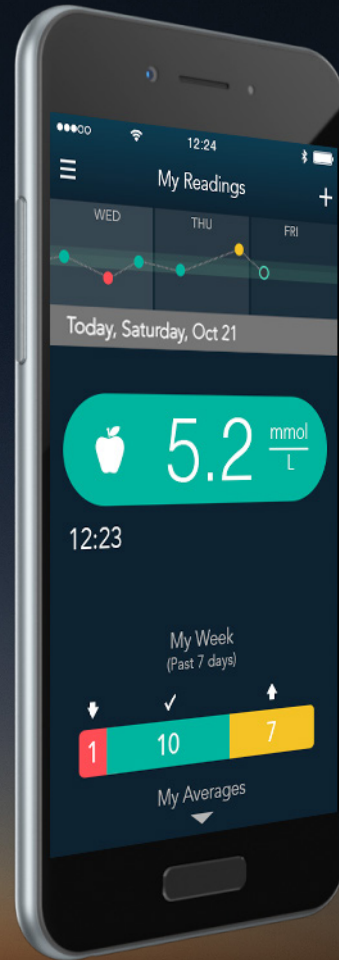
Always consult with your healthcare provider prior to making changes to treatment, diet or exercise.

© Copyright 2017 Ascensia Diabetes Care Holdings AG

Contour®
plus ONE»
Blood Glucose
Monitoring System


diabetes.ascensia.com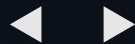

The **smartLIGHT** feature gives you instant feedback on your blood glucose readings.

Contour[®]
plus **ONE**[®]
Blood Glucose
Monitoring System

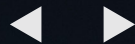
1.

smartLIGHT Feature

After you have monitored your blood glucose and if the Target Light setting is on, the test strip port light displays a color representing your reading value compared to the Before Meal, After Meal, or Overall Target Range.

Contour[®]
plus ONE[»]
Blood Glucose
Monitoring System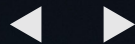

2.


smartLIGHT Feature

If your blood glucose reading is in target, the smartLIGHT feature is green and the meter beeps.

In Target

Contour[®]
plus **ONE**_»
Blood Glucose
Monitoring System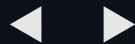

3.


smartLIGHT Feature

If your blood glucose reading is below target, the smartLIGHT feature is red and the meter beeps.

Below Target

Contour[®]
plus ONE[»]
Blood Glucose
Monitoring System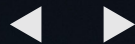

4.


smartLIGHT Feature

If your blood glucose reading is above target, the smartLIGHT feature is yellow and the meter beeps.

Above Target

Contour[®]
plus ONE[»]
Blood Glucose
Monitoring System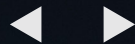

5.

smartLIGHT Feature

If you have not selected a Meal Marker, the smartLIGHT feature will indicate your blood glucose test reading compared to your Overall Target Range.

Contour[®]
plus ONE[»]
Blood Glucose
Monitoring System

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**Contour[®]
plus ONE[™]**
Blood Glucose
Monitoring System

diabetes.ascensia.com

Contour[®]

diabetes solutions