



# Contour<sup>®</sup> plus ONE<sup>»</sup>

Blood Glucose  
Monitoring System

Using the  
**smartLIGHT™** feature on the  
**CONTOUR®PLUS ONE**  
smart meter




Contour®  
plus ONE»  
Blood Glucose  
Monitoring System

Always consult with your healthcare provider prior to making changes to treatment, diet or exercise.

© Copyright 2017 Ascensia Diabetes Care Holdings AG


diabetes.ascensia.com

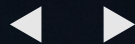

The **smartLIGHT** feature gives you instant feedback on your blood glucose readings.




Contour<sup>®</sup>  
plus **ONE**<sup>®</sup>  
Blood Glucose  
Monitoring System

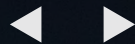


1.

## smartLIGHT Feature

After you have monitored your blood glucose and if the Target Light setting is on, the test strip port light displays a color representing your reading value compared to the Before Meal, After Meal, or Overall Target Range.

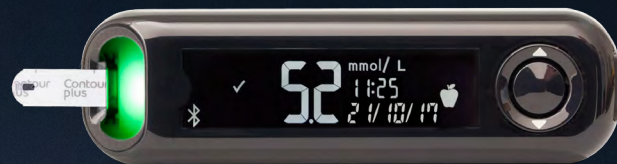

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

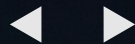

2.


## smartLIGHT Feature

If your blood glucose reading is in target, the smartLIGHT feature is green and the meter beeps.

In Target



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

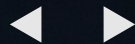

3.


### smartLIGHT Feature

If your blood glucose reading is below target, the smartLIGHT feature is red and the meter beeps.

Below Target



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

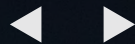

4.


### smartLIGHT Feature

If your blood glucose reading is above target, the smartLIGHT feature is yellow and the meter beeps.

Above Target



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

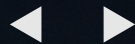

5.

### smartLIGHT Feature

If you have not selected a Meal Marker, the smartLIGHT feature will indicate your blood glucose test reading compared to your Overall Target Range.




Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



Talk to your Healthcare Professional and refer to the product user guide if you have questions about the **CONTOUR® PLUS ONE** smart meter and the **CONTOUR™ DIABETES app**.

For more information, and to find out whether these products are available in your country, visit [diabetes.ascensia.com](http://diabetes.ascensia.com).

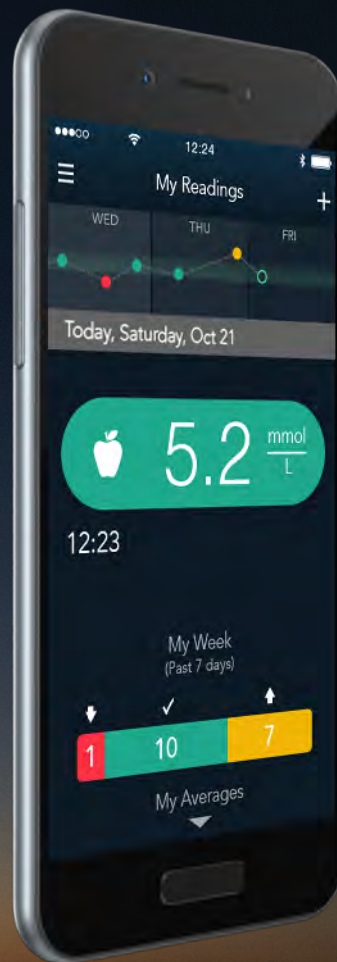

Always consult with your healthcare provider prior to making changes to treatment, diet or exercise.

Get the app



Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks herein is under license. Google play and the Google Play logo are trademarks of Google Inc.

Ascensia, the Ascensia Diabetes Care logo and Contour are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

Date of preparation: February 2018 code: G.DC.12.2017.52593

© Copyright 2017 Ascensia Diabetes Care Holdings AG

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System

[diabetes.ascensia.com](http://diabetes.ascensia.com)



# Contour<sup>®</sup>

diabetes solutions